## UYARILAR

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



**WARNINGS**

- ▶ Before operating the device, read the device user manual completely.
- ▶ It should be used only if deemed appropriate by the specialist physician and with a prescription.
- ▶ The product is in sterile package. Open and torn packages should not be used.
- ▶ The healthcare personnel who will carry out the procedure must comply with the minimum hygienic conditions until the sterile package of the product is opened, and the sterile environment conditions after the package is opened.
- ▶ The Classic Vacuum Assisted Closure Set and the Wound Closure set with Irrigation are suitable for use with TOPIVAC® devices and components. Use with other types of devices may cause undesirable results.
- ▶ Leaving the set on the wound for too long may cause different lesions. The recommended dressing time is 1-3 days.
- ▶ After dressing renewal, used set components should be placed in medical waste bags and disposed of in accordance with infected medical waste procedures.

**⚠** When applying the medical foam on open bone, open tendon, graft, and flap, use antiseptic gauze dressing or materials suitable for this procedure, which will prevent adhesion and pain formation.




**APPLICATION OF TOPIVAC VACUUM-ASSISTED WOUND CARE DRESSING SET**

- 1- Determine the closure set of the most appropriate size and shape for the wound.
- 2- Put on sterile gloves before starting the dressing to avoid infecting the wound bed.
- 3- Clean the wound bed in accordance with the advice of your doctor and with the method determined. If necessary, debride all necrotic tissue.  
Make sure that there is no bleeding in the wound by visual and touch, and that there are no open vessels or sensitive structures in the wound bed.
- 4- Open the package by pulling the package of Topivac dressing set from the short edge with both hands and separating it.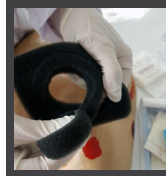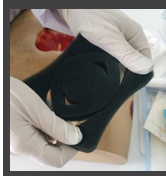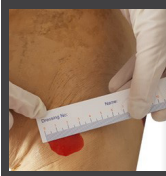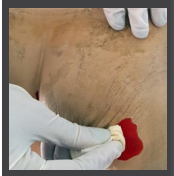





- 5- Wipe the healthy tissue around the wound with a cleaning wipe or tincture of benzoin or etc. In the application of the dressing, the healthy skin area where the drape is adhered should be oil-free and clean.
- 6- Measure the size of the wound with the help of ruler that comes out of the set, and shape the medical foam to fit the wound bed with sterile scissors.
- 7- Place the dressing foam prepared according to the wound bed on the wound bed.

**⚠** Do medical foam sizing and cutting procedures away from the wound area.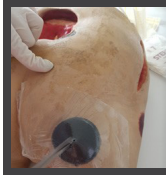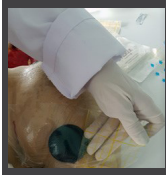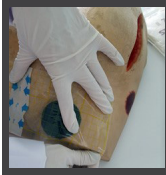

- 8- Cut the drape that comes out of the set in sizes suitable for the dressing. Place the drape, centered on the wound dressing so that there is no vacuum leakage on to the medical foam and stick it.
- 9- Remove the protective film on the drape.
- 10- Cut the drape with sterile scissors to center the foam placed on the wound bed and make a 1-2 cm² hole.

- 11- Holding the trackpad (circular drape) with both hands, open the film tape on the part to be adhered to. By centering the suction cup on the hole cut on the drape, stick the trackpad on the drape together with the suction cup in a way that there is no vacuum leakage.
- 12- Remove the film layer on the trackpad.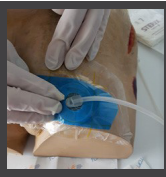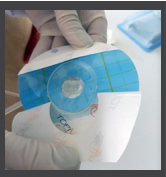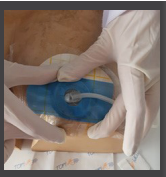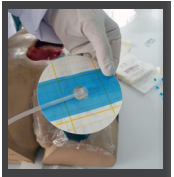

- 13- To avoid vacuum leakage, make sure that the trackpad and the drape stuck to the skin stick well by rubbing them with your hand.
- 14- Connect the thick vacuum hose on the dressing to the hose coming from CANISTER.

**⚠** In this dressing method, there should be no vacuum leakage. If there is a vacuum leakage in the dressing, it becomes difficult to get the desired result in the treatment.

- 15- When using the MultiDRESS Irrigation set, drill 2 holes.
- 16- In irrigation sets, the thin hose with suction cups and the extension hose from the dressing are connected to the valve of the Multicase Irrigation Cup.
- 17- Topical vacuum wound therapy is started with Topivac devices and components.

MultiDRESS Irrigation Closure Set:		
1.	Medical Foam	1 Piece
2.	Drape	1 Piece
3.	Ruler	1 Piece
4.	Vacuum Line Hose and Trackpad (Thick)	1 Piece
5.	Irrigation Hose and Trackpad	1 Piece
6.	Irrigation Extension Hose (Thin)	1 Piece
7.	User Manual	1 Piece

MultiDRESS Classic Closure Set:		
1.	Medical Foam	1 Piece
2.	Drape	1 Piece
3.	Ruler	1 Piece
4.	Vacuum Line Hose and Trackpad (Thick)	1 Piece
5.	User Manual	1 Piece